

IMPROVED



Self-sticking tender tape

SPITA ResQ-plast®

PROFESSIONAL

- Doesn't stick to skin or hair
- Permanently breathable
- Self-sticking
- Prevents allergy to plaster




- Permanently stretchable
- Washable
- Resists oil, grease and dirt
- Works even under water


CAN BE REMOVED PAINLESSLY



FAST AND EASY TO USE:



To make a dressing, fold the plaster several times onto itself to get the desired thickness. The dressing must be large enough to cover the bleeding wound.

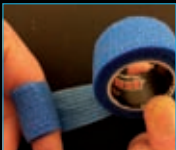

Place the dressing onto the wound and stretch **SPITA ResQ-plast** a little and wrap it securely. Wrap 4-5 layers from left to right (or right to left) and tear the tape. Press the loose end of the tape firmly onto itself to secure it.

www.spita.eu

SPITA
ResQ-technology

Self-sticking tender tape **ResQ-plast**



The versatile and skin friendly self-sticking plaster!

Because **SPITA ResQ-plast** only sticks to itself, it is easy to use on a wet and/or dirty skin. In case of severe bleeding injuries you may apply **SPITA ResQ-plast** direct onto the wound to stop the bleeding. When you stretch **SPITA ResQ-plast** a little, while you wrap it around, it will stick immediately onto itself. The plaster remains flexible at all times and it will not hinder the moves of your joints.

SPITA ResQ-plast can be applied under water (!) and will not come off during swimming or when you take a shower. **SPITA ResQ-plast** is therefore extremely useful with jobs where (a lot of) water or fluids are used (dish water, cooking, washing, etc.) and where your hands often will become wet (chefs, butchers, plumbers, firemen, etc.)

SPITA ResQ-plast is excellent for:

- ✓ Injuries to fingers, toes, etc.
- ✓ Cuts, abrasions, lacerations
- ✓ Blisters
- ✓ Compression bandage
- ✓ Joint injuries
- ✓ Torn nails
- ✓ Wet, dirty or greasy skin
- ✓ Animals / Pets
(doesn't stick to the fur)

User instructions to wrap on finger tip:



Apply **SPITA ResQ-plast** over the tip of the finger.



Wrap **SPITA ResQ-plast** from back to front.



After 4-5 wraps tear **SPITA ResQ-plast** and press end of tape firmly onto itself.



This is also an excellent method to protect your finger(s) against sharp objects.

WARNING:

When **SPITA ResQ-plast** is wrapped too tight, it can hinder the blood circulation. If this occurs please unwrap the material and wrap it less tight.

TIP:

SPITA ResQ-plast has a very long shelf-life. Keep **SPITA ResQ-plast** in the blister pack to prevent it from becoming dirty.

Dimensions:

"standard" : 25 mm (w) x 4,5 mtr (l)
"wide" : 50 mm (w) x 4,5 mtr (l)

Available in the following colours:

RED **BLUE** **BEIGE** **BLACK** **CAMOUFLAGE** **TURQUOISE** **WHITE** **GREEN**

Certified for medical use by:



Please visit our website for further information:

www.spita.eu

SPITA Smart Products B.V.
Industrieweg 18-1, 3846 BD Harderwijk (NL)
Tel.: +31 (0)341 - 41 49 93, E-mail: sales@spita.eu

Other ResQ-products:



This product is provided without warranty, either expressed or implied, of fitness for a specific purpose. Because conditions and methods of use are beyond the manufacturer's control, neither manufacturer nor seller will assume any responsibility for the use or misuse of this product. This includes, but is not limited to, any consequential injuries. **SPITA ResQ-plast** is a trademark of G&G Investments B.V., Zeewolde (NL).

Powered by